




Histofix[®] Substitute of Formaldehyde for clinical diagnosis

Commission Regulation (EU) 605/2014 and amendment No. 2015/491 establishing new rules for classification and labeling of dangerous substances and precautionary statements and use of these substances. Formaldehyde is, among others, one of the substances affected by this new European legislation, so that we recommend the use of alternative substances and decreasing the exposure times of the users to this product.

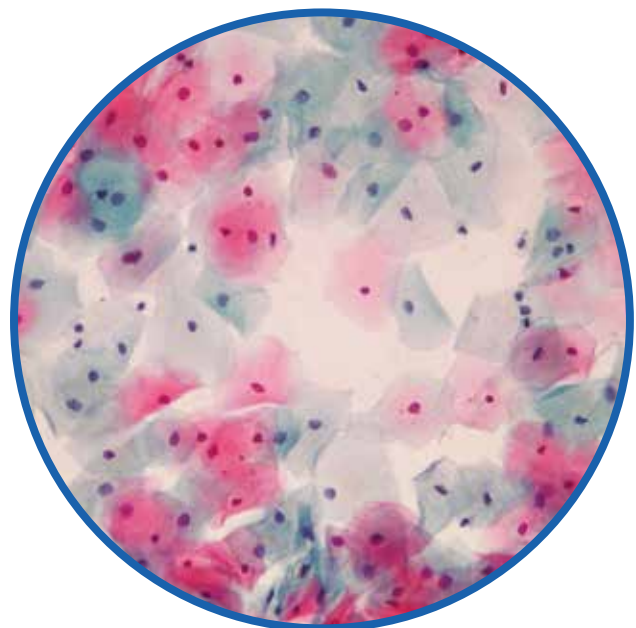

PanReac AppliChem has developed a new product that assists to achieve the objectives of the new regulations: The new Histofix[®] Substitute of Formaldehyde is a fixing agent commonly used in histological techniques, based on glyoxal, which reduces the health hazards for users.

Main advantages

- It is not carcinogenic.
- Flexible. It adapts to different fixation procedures: product concentrated and ready to use
- Convenient and practical packaging (1 and 5 liters) depending on the needs of each user.
- Less pungent odour than formaldehyde.



IP-019EN;201506

Products available:

- Histofix® Substitute of Formaldehyde ready to use, code 257127.
Dilution is not needed.
- Histofix® Substitute of Formaldehyde, code 255805.
Dilute before use according to the instructions below.

For regular fixations

Mix 1 L of Histofix® Substitute of Formaldehyde, 3.1 L of water and 0.9 L of absolute ethanol.

For fixations with alcoholic effect

Mix 1 L of Histofix® Substitute of Formaldehyde, 2 L of water and 1.9 L of absolute ethanol.



Ordering Information

Description	Code	Package
Histofix® Substitute of Formaldehyde for clinical diagnosis	255805.2711	1000 mL
	255805.2714	5 L
Histofix® Substitute of Formaldehyde ready to use for clinical diagnosis	257157.1211	1000 mL
	257157.1214	5 L

Other related products:

Histofix® Preservative in bottles ready for use (code 256 462), is a product that reduces the time of exposure to formaldehyde. Request specific brochure.

IP-019EN

AppliChem GmbH
Ottoweg 4
DE-64291 Darmstadt
Germany
Phone +49 6151 9357-0
Fax +49 6151 93 57-11
service@applichem.com
www.applichem.com

Nova Chimica Srl
Via G. Galilei, 47
I-20092 Cinisello Balsamo
(Milano) Italy
Phone +39 02 66045392
Fax +39 02 66045394
info@novachimica.com
www.novachimica.com

PanReac Quimica SLU
Garraf, 2
Polígono Pla de la Bruguera
E-08211 Castellar del Vallès
Barcelona (Spain)
Phone +34 937 489 400
Fax +34 937 489 401
central@panreac.com
www.panreac.com

